Autolet® Impression

EN INSTRUCTIONS FOR USE

Autolet Impression is an advanced lancing device, which provides maximum comfort and control when obtaining a blood sample.

Autolet Impression must not be used in multi-patient clinical environments.

Recommended for use with Unilet brand lancets.

Please read through these instructions carefully before using Autolet Impression for the first time, and retain them for future reference.

Components

- a. Depth adjuster
- b. Depth adjuster collar
- c. Depth adjuster assembly
- d. Penetration depth indicator
- e. Device body
- f. Release button
- g. Force adjustment indicator
- h. Force adjustment ring
- i. Device sleeve
- j. Clear alternate site end cap

Comfort Zone Technology™

Comfort Zone Technology increases comfort when lancing by targeting the sensation and perception of pain. This unique approach disguises the lancing action to provide a more comfortable sample.

Force Adjustment Feature

Force Adjustment is designed to enable a user to alter the force of the lancet allowing people with tougher skin to easily acquire a suitable blood sample. This may be used as an alternative to increasing the penetration depth and can help achieve a suitable sample in a more comfortable way.

Penetration Depth Settings

Autolet Impression features 7 penetration depth settings to achieve maximum comfort when sampling. 0 is the minimum depth and 6 is the maximum.

Combined, these features provide users with the maximum level of lancing comfort.

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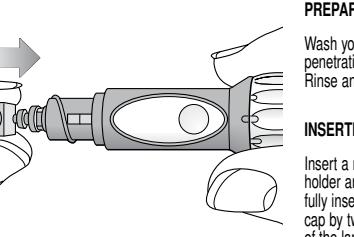
Fig.1

PREPARING THE SITE

Wash your hands and the site of penetration with warm, soapy water. Rinse and dry thoroughly.

INSERTING THE LANCET

Insert a new lancet into the lancet holder and push down firmly until it is fully inserted. Remove the protective cap by twisting it away from the body of the lancet. (Fig.1)

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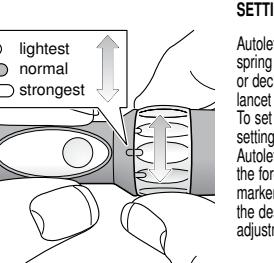
Fig.5

SETTING THE SPRING FORCE

Autolet Impression has 3 unique spring force settings that increase or decrease the force at which the lancet penetrates the sampling site. To set the device to the desired setting, hold the main body of the Autolet Impression and then twist the force adjustment ring until the marker on the body is in line with the marker on the force adjustment ring. (Fig.5)

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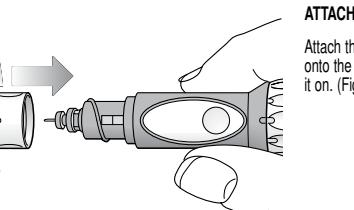
Fig.2

ATTACHING THE DEPTH ADJUSTER

Attach the depth adjuster assembly onto the Autolet Impression by twisting it on. (Fig.2)

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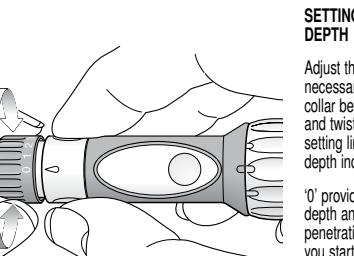
Fig.3

SETTING THE PENETRATION DEPTH

Adjust the penetration depth as necessary. Hold the depth adjuster collar between thumb and forefinger and twist until the required depth setting lines up with the penetration depth indicator.

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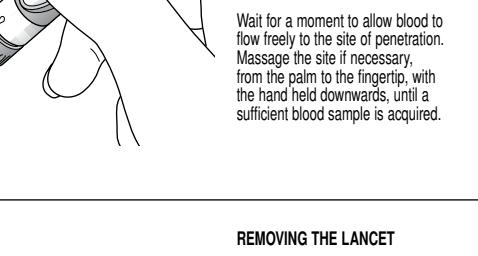
Fig.6

OBTAINING THE SAMPLE

Hold the front end of Autolet Impression firmly against the sample site. (Fig.6)

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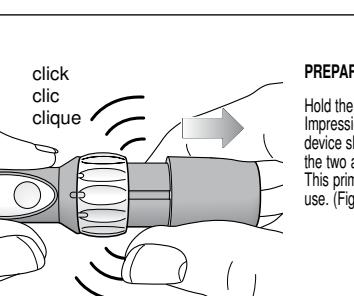
Fig.4

PREPARING TO TAKE THE SAMPLE

Hold the main body of Autolet Impression with one hand and the device sleeve with the other and pull the two apart until you hear a click. This primes the device ready for use. (Fig.4)

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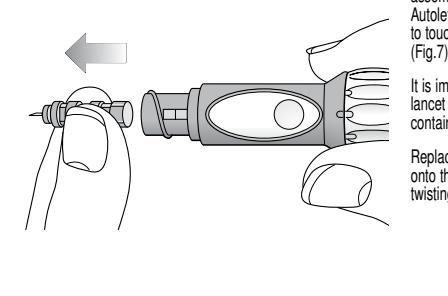
Fig.7

REMOVING THE LANCET

Twist off the depth adjuster assembly to remove it from the Autolet Impression taking care not to touch the exposed lancet needle. (Fig.7)

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Substitua o conjunto regulador de profundidade na Autolet Impression rodando-o.

ALTERNATE SITE BLOOD SAMPLING

Alternate sites, such as the arm, have fewer nerve endings than the fingertip so you may find that acquiring a blood sample from the arm may be less painful than using a fingertip. Before you begin testing on an alternate site, you should read the owner's booklet that came with your blood glucose meter and talk to your healthcare professional.

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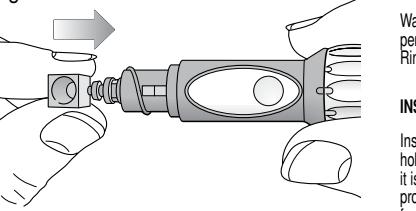
Fig.8

PREPARING THE SITE

Wash your hands and the site of penetration with warm, soapy water. Rinse and dry thoroughly.

INSERTING THE LANCET

Insert a new lancet into the lancet holder and push down firmly until it is fully inserted. Remove the protective cap by twisting it away from the body of the lancet. (Fig.8)

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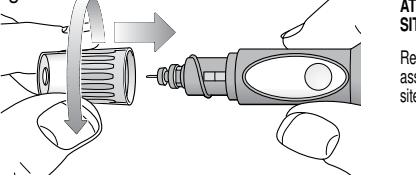
Fig.9

ATTACHING THE ALTERNATE SITE END CAP

Replace the normal depth adjuster assembly with the clear alternate site end cap. (Fig.9)

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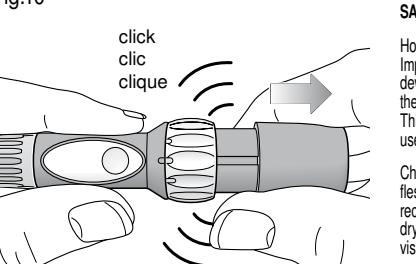
Fig.10

PREPARING TO TAKE THE SAMPLE

Hold the main body of Autolet Impression with one hand and the device sleeve with the other and pull the two apart until you hear a **click**. This primes the device ready for use. (Fig.10)

Choose a penetration site on a soft, fleshy area of your arm, or other recommended site, that is clean and dry, away from bone, and free of visible veins and hair.

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Rub the test site vigorously for a few seconds until you feel it getting warm. This will bring blood to the surface of your arm.

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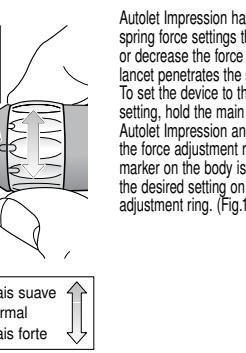
Fig.11

SETTING THE SPRING FORCE

Autolet Impression has 3 unique spring force settings that increase or decrease the force at which the lancet penetrates the sampling site. To set the device to the desired setting, hold the main body of the Autolet Impression with the force adjustment ring until the marker on the body is in line with the desired setting on the force adjustment ring. (Fig.11)

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AUTOLET IMPRESSION CLEANING AND MAINTENANCE

CLEANING AND DISINFECTING

Use mild soap and water to wipe the outside of Autolet Impression with a soft cloth. Once a week, wash the depth adjuster assembly and the alternate site testing cap in mild soap and water. Allow to dry before storing. Do not immerse Autolet Impression in water.

SAFETY INFORMATION

Lancets are sterile, single use, disposable device.

Autolet Impression is for use with one end user. Choose a different penetration site each time you test. Repeated penetrations in the same spot may cause soreness and calluses. After sampling, wipe the penetration site with a dry, clean tissue. If it continues to bleed apply gentle pressure with a plaster to protect it from dirt or infection.

WARRANTY INFORMATION

Autolet Impression is guaranteed against manufacturing defects for two years from date of purchase. In the unlikely event of failure, please contact your local Owen Mumford. Returning the device will not affect your statutory rights.

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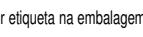
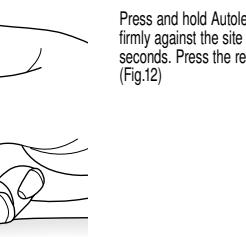
Fig.12

OBTAINING THE SAMPLE

Press and hold Autolet Impression firmly against the site for a few seconds. Press the release button. (Fig.12)

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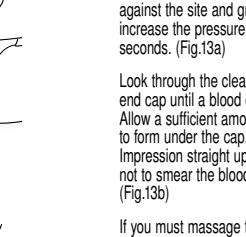
Fig.13a


Continue to hold Autolet Impression against the site and gradually increase the pressure for a few seconds. (Fig.13a)

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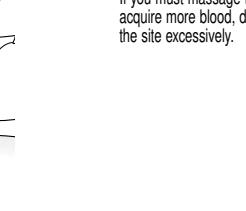Fig.13b

Look through the clear alternate site end cap until a blood drop forms. Allow a sufficient amount of blood to form under the cap. Lift Autolet Impression straight up being careful not to smear the blood sample. (Fig.13b)

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If you must massage the area to acquire more blood, do not squeeze the site excessively.

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Autolet® is a registered trademark and Comfort Zone Technology™ is a trademark of Owen Mumford Ltd.

Unilet® is a registered trademark of Owen Mumford.

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Owen Mumford Inc.
Owen Mumford GmbH.